



JDENTALCARE


just smile



MADE IN ITALY

Dental Implant

JD ZYGOMA[®]

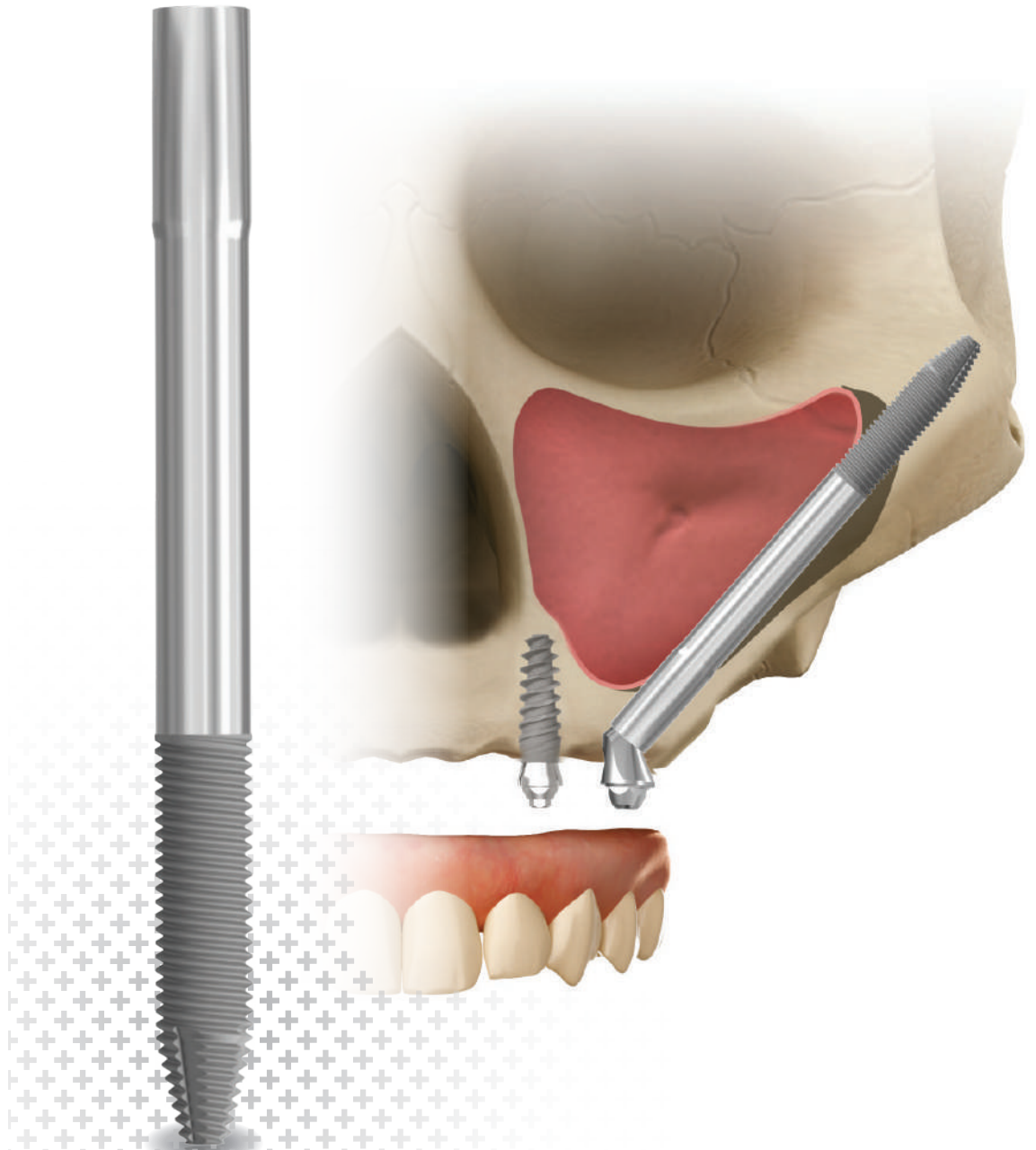

A Graftless solution

The implant for severe resorption
in maxillary bone.

JD ZYGOMA[®]

A GRAFTLESS SOLUTION.

Titanium Grade 4

Implant body \varnothing 4.3 mm

High primary stability for the Immediate function

Apply to your patients a temporary prosthesis immediately after the surgery. Thanks to the multicortical anchorage and the special design of the implant apex you will easily obtain an high primary stability.

Implant collar \varnothing 4.5 mm


Greater prosthetic and surgical flexibility

Take advantage of the advanced implant body design with a straight and polished neck with implant lengths from 30 mm to 52.5 mm and internal hexagonal connection compatible JDEvolution Plus[®].

A graftless treatment solution

Avoid complex bone grafting procedures with the new JDZygoma[®] implant. This advanced implant concept is designed for getting a perfect anchorage in the zygomatic bone.


Tip Diameter 2.0 mm

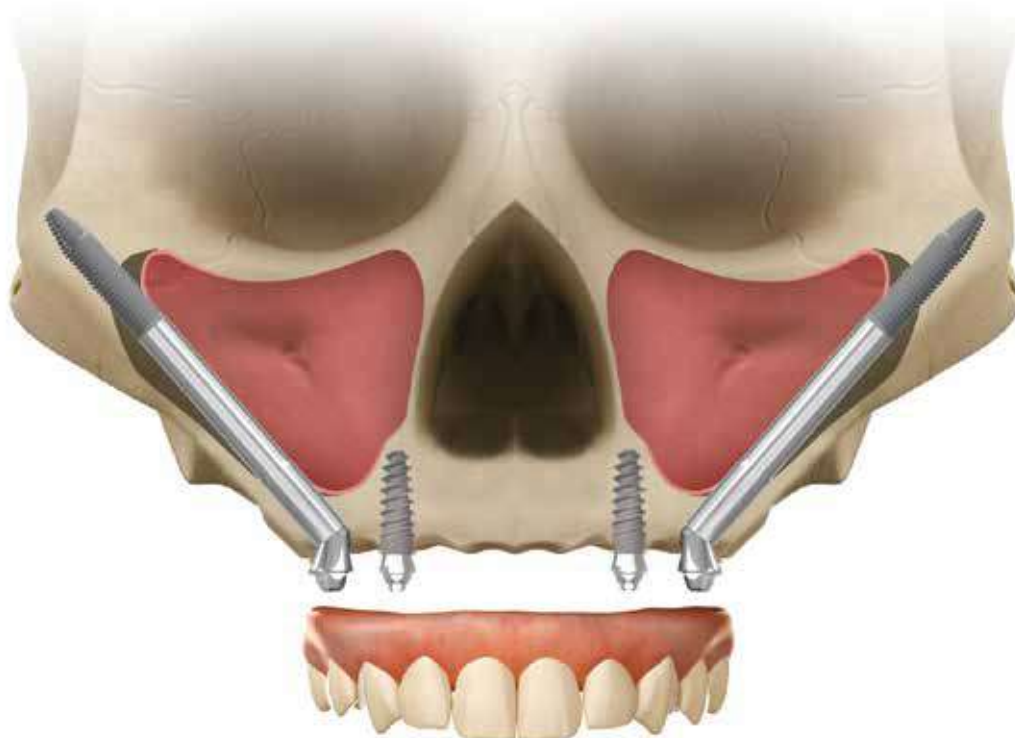

PRODUCT SPECIFICATIONS.


IMPLANT DIAMETER	TIP DIAMETER	IMPLANT LENGTH
Ø 4.3	2.0	30
		35
		37.5
		40
		42.5
		45
		47.5
		50
		52.5

Note:
All dimensions are expressed in millimetres.

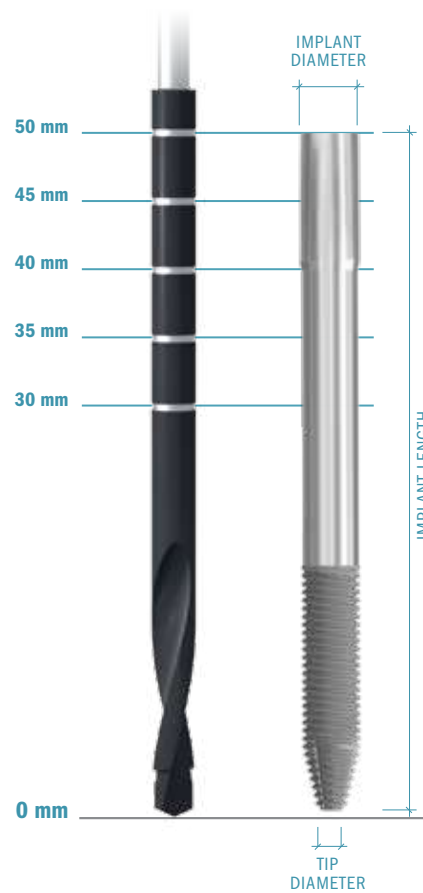

SITE PREPARATION SEQUENCE.

Make entrance mark with the round bur drill and continue with the drill Ø 2.8 mm until it penetrates the outer cortical layer of the zygomatic bone. Continue with the drill Ø 3.2 mm and if the bone is hard, complete the osteotomy with the drill Ø 3.6 mm.

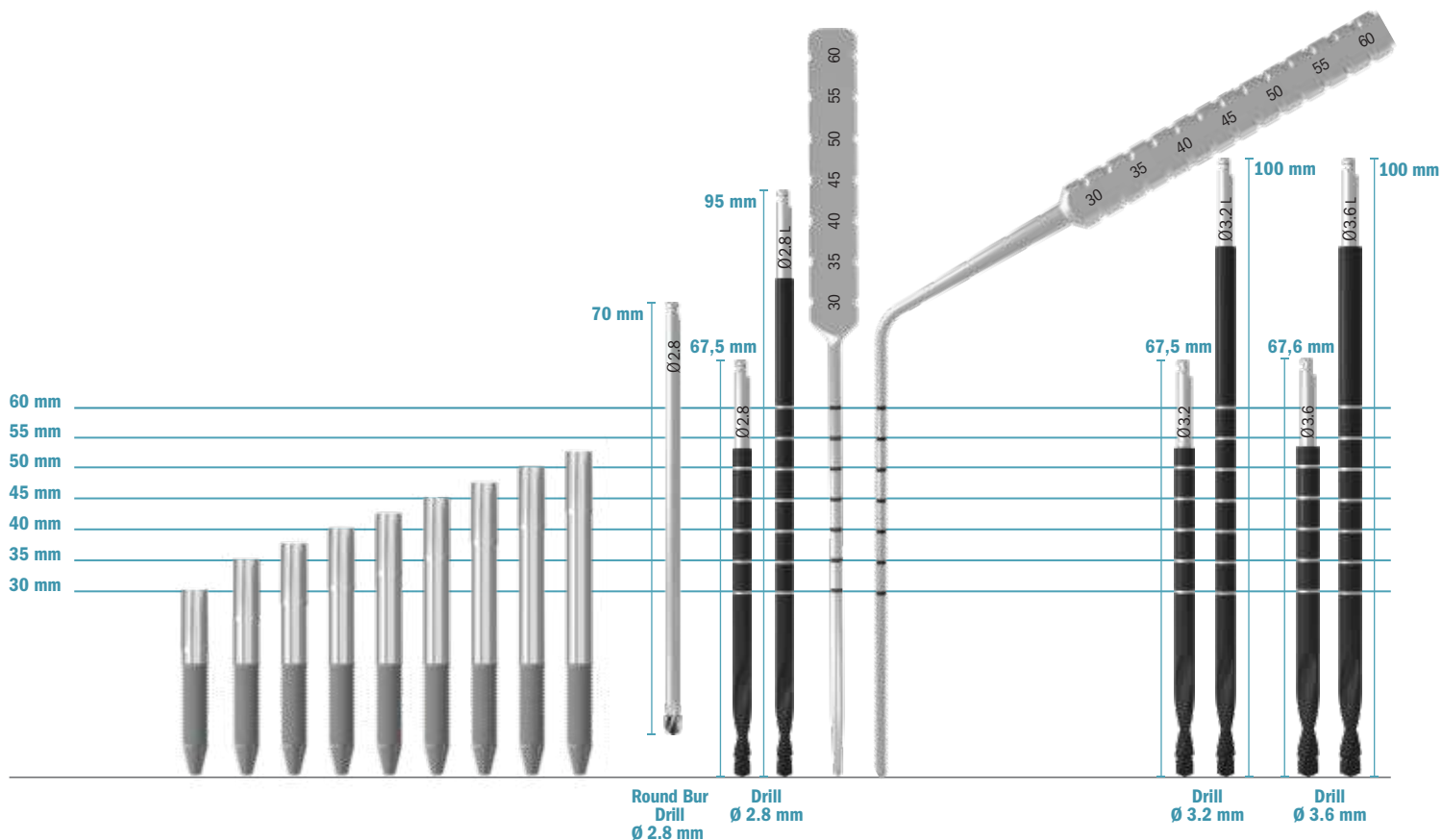

PRODUCT CATALOGUE.

Implants:

ZY43300:	JDZygoma® Ø 4.3 L 30
ZY43350:	JDZygoma® Ø 4.3 L 35
ZY43375:	JDZygoma® Ø 4.3 L 37.5
ZY43400:	JDZygoma® Ø 4.3 L 40
ZY43425:	JDZygoma® Ø 4.3 L 42.5
ZY43450:	JDZygoma® Ø 4.3 L 45
ZY43475:	JDZygoma® Ø 4.3 L 47.5
ZY43500:	JDZygoma® Ø 4.3 L 50
ZY43525:	JDZygoma® Ø 4.3 L 52.5




Drills and Depth probe:

JDDRZSF28	Round Bur drill Ø 2.8 JDZygoma®
ZJDDR28	Drill Ø 2.8 JDZygoma®
ZJDDR28L	Long Drill Ø 2.8 JDZygoma®
ZJDDR32	Drill Ø 3.2 JDZygoma®
ZJDDR32L	Long Drill Ø 3.2 JDZygoma®
ZJDDR36	Drill Ø 3.6 JDZygoma®
ZJDDR36L	Long Drill Ø 3.6 JDZygoma®
JDZPRS	60mm straight depth probe JDZygoma®
JDZPR	60mm depth probe JDZygoma®



Conical Abutment:

EVCA4545C:	45° angulated Conical Abutment H 4.5 JDEvolution®Plus+	Torque recommended 30 Ncm
EVCA6050C:	60° angulated Conical Abutment H 5.0 JDEvolution®Plus+	
EVCAPS:	Prosthetic screw Conical Abutment JDEvolution®Plus+	Torque recommended 15 Ncm
EVCASA:	Screw for angulated Conical Abutment JDEvolution®Plus+	Torque recommended 30 Ncm
EVCAPSA	Prosthetic screw angulated for Conical Abutment	Torque recommended 15 Ncm
EVCAICOTC:	Impression coping open tray Conical Abutment JDEvolution®Plus+	
EVCAICCTC	Impression coping closed tray Conical Abutment	
EVCAHCL:	Healing cap Conical Abutment H 6.0 JDEvolution®Plus+	
EVCAHC:	Healing cap Conical Abutment H 3.5 JDEvolution®Plus+	
EVCAHCB:	Healing cap peek Conical Abutment H 3.5 JDEvolution®Plus+	
EVCAGPAEC:	GP engaging Conical Abutment JDEvolution®Plus+	
EVCAGPANEC:	GP non engaging Conical Abutment JDEvolution®Plus+	
EVCATANEC:	Temporary cylinder non engaging Conical Abutment JDEvolution®Plus+	Torque recommended 15 Ncm
EVCATANEWC:	Temporary cylinder non engaging for welding Conical Abutment JDEvolution®Plus+	
EVCAWANEC:	Wax up cylinder non engaging Conical Abutment JDEvolution®Plus+	
EVCAAR	Abutment replica Conical Abutment	

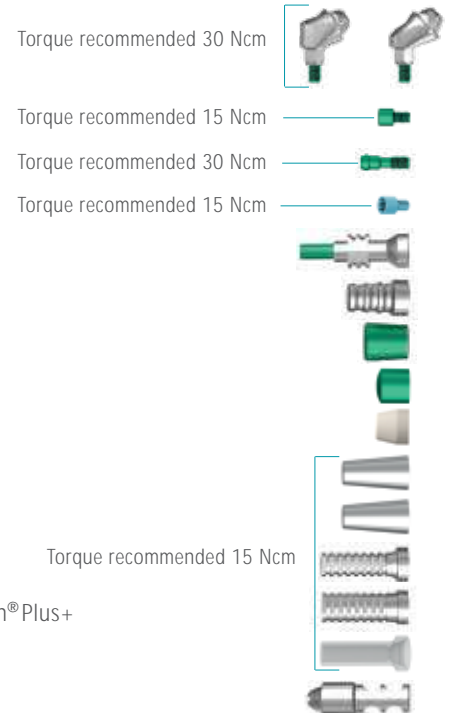

JDScanBody, Interface and Abutment Replica CAD CAM for Conical abutment:

EVCASBCEC:	JDScanBody for Conical Abutment JDEvolution®Plus+
EVCAITEC:	Interface CAD CAM engaging for Conical Abutment H 7.5
EVCAITC:	Interface CAD CAM for Conical Abutment H 7.5
EVCATIBC:	Interface CAD CAM for Conical Abutment H 5.0
EVCAARC:	Abutment Replica CAD CAM for Conical Abutment



Healing Abutment:

EVHA43:	Healing Abutment Ø 4.0 H 3.0 JDEvolution®Plus+
EVHA45:	Healing Abutment Ø 4.0 H 5.0 JDEvolution®Plus+
EVHA47:	Healing Abutment Ø 4.0 H 7.0 JDEvolution®Plus+
EVHA53:	Healing Abutment Ø 5.0 H 3.0 JDEvolution®Plus+
EVHA55:	Healing Abutment Ø 5.0 H 5.0 JDEvolution®Plus+
EVHA57:	Healing Abutment Ø 5.0 H 7.0 JDEvolution®Plus+



Implant and prosthetic drivers:

EVID:	Implant driver JDEvolution®Plus+
EVIDL:	Implant driver long JDEvolution®Plus+
EVSDPF20:	Screwdriver for torque wrench L20 JDEvolution®Plus+
EVSDPF25:	Screwdriver for torque wrench L25 JDEvolution®Plus+
EVSDCAF	Screwdriver torque wrench Conical Abutment
EVSDP20:	Screwdriver machine prosthetic L20 JDEvolution®Plus+
EVSDP25:	Screwdriver machine prosthetic L25 JDEvolution®Plus+
EVSDP30:	Screwdriver machine prosthetic L30 JDEvolution®Plus+
EVSDCA	Screwdriver machine Conical Abutment
EVSDPF25A	Screwdriver screw angulated for torque wrench L25
EVSDPF30A	Screwdriver screw angulated for torque wrench L30
EVSUD	JD Surgical Driver



JD Zygoma Surgical Kit:

ZSKIT	JD Zygoma Surgical Kit
-------	------------------------



JDENTALCARE

www.jdentalcare.com